操作简便 Easy to use

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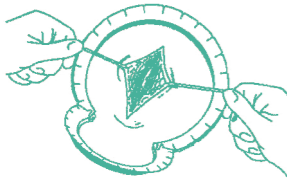
Only three steps to setup: place the retractor ring at the right position-pull the tissue outwards by retractor's hooks-fix retractors on the ring.

自由牵开、显露 Flexibility in retraction and exposure

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According to different surgery demands to select retractors and adjust their position to optimize the exposure of surgical field.

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Place the retractor ring around the incision, Use the hook of the retractor to pull the tissue and then tighten the retractor to the appropriate position on the ring.

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Reposition or add new retractors if needed.

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When using the device to assist suturing, position and tighten the retractor along the axis of the incision to fix the position of the incision.

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Bonnie Medical Device Technology (Changzhou)Co.,Ltd.

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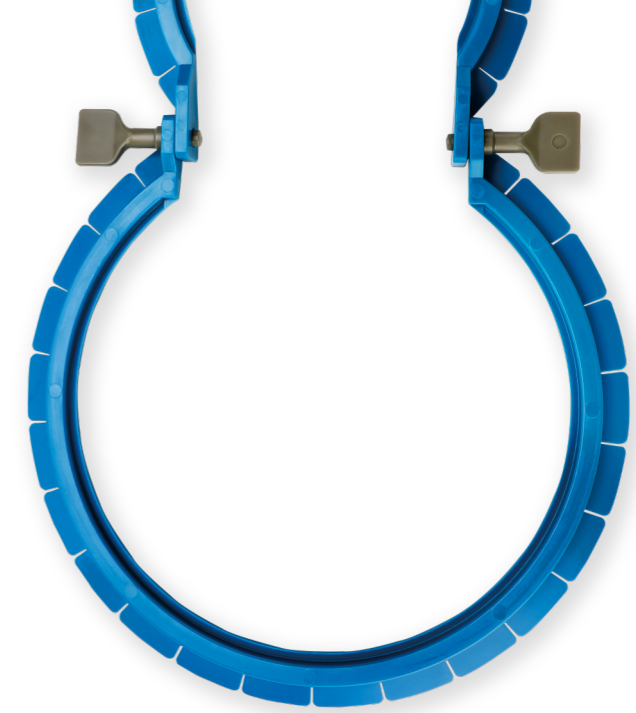
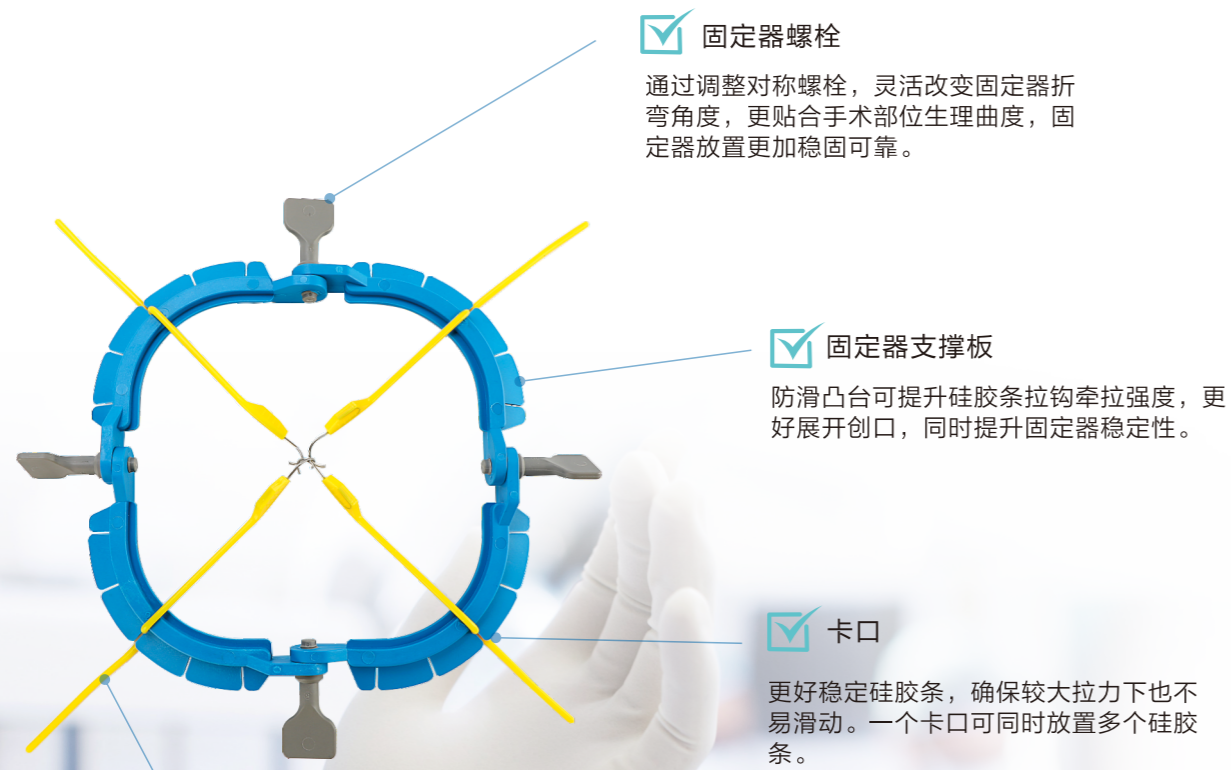
The disposable surgery retractor system operates manually with self-locking function. The system applies to temporarily separate tissues or anatomical structures to increase exposure of internal tissues or organs in surgical field. The retractor equipped with various heads such as blunt, sharp fenestral, shape and so on.

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